MERCATOR MEDICAL



MERCATOR simple vinyl

The instruction below should be used in conjunction with detailed information on the packaging.

Short description of the product

Vinyl protective gloves, powder-free or powdered, non-sterile for disposable use, for minimal risk

Full description of the product	
Reference number:	: RP20018001 - 05 (1537)
	RP20017001- 05 (1537) lub (001)
	RP20016001 - 05 (1537) lub (001)
	RP21015001 - 05 (1537) lub (001)
Raw material	: vinyl
Cuff	: beaded
Colour	: black / blue / clear
Shape	: ambidextrous, fitting either hand
Size range	: XS (5-6), S (6-7), M (7-8), L (8-9), XL (9-10)
Quantity in packaging	: 100 pcs. by weight
Shelf life	: 5 years for powdered gloves
	3 years for powder-free gloves

Storage instructions

It is recommended to store the gloves in dry place, in the temperature of 5-35°C and to protect them against direct sunlight and fluorescent light. It is recommended to protect gloves against humidity.

Keep the gloves in a distance of not less than 1m from heating devices, sources of fire and ozone.

Do not keep in direct vicinity of solvents, oils, fuels and lubricants.

Food contact

Gloves are marked with food contact symbol \checkmark and comply with the requirements of Regulation (EU) No 10/2011, European Regulation (EC) No 1935/2004 and with Regulation (EC) No 2023/2006 on Good Manufacturing Practice.

PPE classification & compliance

Gloves are classified as Personal Protective Equipment – category I for minimal risk according to the Regulation 2016/425.

Compliance with standards: EN 420:2003+A1:2009

Declaration of Conformity can be found under below web address:

https://mercatormedical.eu/products/rekawice/gospodar cze_i_ochronne/MERCATOR-simple-vinyl-PE

https://mercatormedical.eu/products/rekawice/gospodar cze_i_ochronne/MERCATOR-simple_vinyl_blue

https://mercatormedical.eu/products/rekawice/gospodar cze_i_ochronne/MERCATOR-simple-vinyl-PF-black

https://mercatormedical.eu/products/rekawice/gospodar cze_i_ochronne/MERCATOR-simple-vinyl-PP

Intended use

These are non-sterile protective gloves, intended for use in hygienic and sanitary works in public places, for maintaining cleanness in facilities, utility rooms and home, professional cleaning services using cleaning materials of weak action, gardening, agriculture, food processing and food contact, for use in household and mechanical works with risk only superficial mechanical injury. Classified as Personal Protective Equipment category I for minimal risk. Their design and labeling corresponds to the requirements of the Regulation 2016/425 on Personal Protective Equipment. Gloves should be used solely according to their intended purpose.

Precautions and indications for use

Dry hands before putting the gloves on. Before usage, inspect the gloves for any defect or imperfections. If the gloves get punctured, torn or broken during their use, take them off and put on the new ones. Avoid using gloves dirty in the inside as they may cause irritation leading to skin inflammation or more serious damages. The gloves should not be used in contact with open fire and to protect against any sharp tools. The gloves are not intended for welding, electric shock protection, ionizing radiation or from the effect of hot or cold objects.

Components / hazardous components

Some gloves may contain components known to be a possible cause of allergy for person allergic to them, who may develop contact irritation and/or allergic reaction. Gloves made from natural rubber latex may cause allergic reactions, including anaphylactic responses. In the case of an allergic reaction, seek medical assistance immediately.


Disposal

Used gloves should be disposed of in accordance with local regulation. Gloves should be buried or burned under controlled conditions.


Manufacturer

MERCATOR MEDICAL S.A. ul. H. Modrzejewskiej 30 31-327 Cracow, Poland www.mercatormedical.eu

MERCATOR MEDICAL

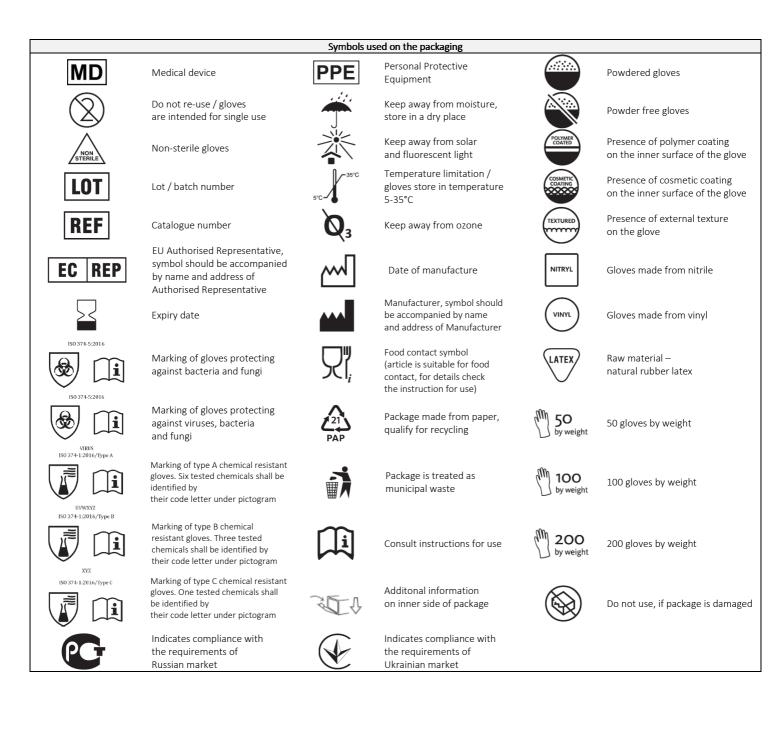

MERCATOR MEDICAL

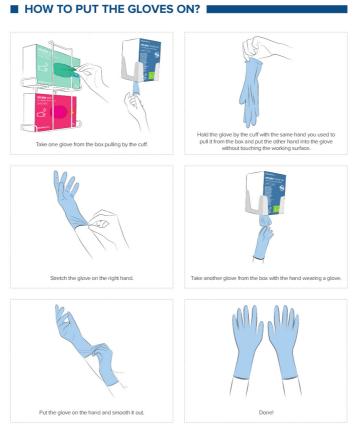

■ HOW TO TAKE THE GLOVES OFF?

